### Fitting instructions

- Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.
- Position the respirator under your chin with the nosepiece up.
- Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears.
- Place the fingertips of both hands at the top of the metal nosepiece. Mould the nosepiece to the shape of your nose by pushing inwards while moving your fingertips down both sides of the nosepiece. Pinching the nosepiece using only one hand may result in less effective respirator performance.
- The seal of the respirator on the face should be fit-checked prior to wearing in the work area.
  - Cover the front of the respirator with both hands, being careful not to disturb the position of the respirator.
  - Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust the position of the respirator and/or tension of the straps. Retest the seal. Repeat the procedure until the respirator is sealed properly. If you CANNOT achieve a proper fit DO NOT enter the contaminated area. See your supervisor.

**Note: do not use with beards or other facial hair that may inhibit contact between the face and the edge of the respirator.**

### Warning

- As with the use of any respiratory device, the wearer must first be trained in the proper use of the product.
- This product does not protect the wearer against oil-based aerosols, gases or vapours, solvents from paint spray operations or in atmospheres containing less than 19.5% oxygen.
- Use only in adequately ventilated areas containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are immediately dangerous to life or health.
- Leave the area immediately if:
  - breathing becomes difficult;
  - dizziness or other distress occurs.
- Discard and replace the respirator if it becomes damaged, breathing resistance becomes excessive, or at the end of the shift.
- Never alter or modify this device.
- Failure to follow all instructions and warnings on the use of this respirator during all times of exposure can reduce the respirator effectiveness and may result in sickness or permanent disability.

### Approvals

This product has been shown to meet the Basic Safety Requirements under Article 10 and 11B of the European Community Directive 89/686 and is thus CE marked. The product has been examined at the design stage by Berufsgenossenschaftliches Institut für Arbeitssicherheit, Postf. 2043, Alte Heerstr. 111, D-53757 St. Augustin, Germany (0121).

**3M United Kingdom PLC.**  
3M House, P.O. Box 1, Market Place  
Bracknell, Berks. RG12 1JU  
Tel: 0800 212490

**3M Ireland**  
3M House, Adelphi Centre  
Dun Laoghaire, Co. Dublin  
Tel: 1800 320500

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